

PATIENT INSERT

EPIPEN®

(epinephrine) Auto-Injector 0.3 mg

EpiPen® = one dose of 0.30 mg epinephrine (USP, 1:1000, 0.3 mL)

EPIPEN® JR

(epinephrine) Auto-Injector 0.15 mg

EpiPen® Jr = one dose of 0.15 mg epinephrine (USP, 1:2000, 0.3 mL)



PATIENT INSERT

Pharmacist - Please Dispense this Leaflet with Product

IMPORTANT INFORMATION

The patient and caregiver should read this information carefully before using the EpiPen® or EpiPen® Jr Auto-Injector. Please be prepared! Read the entire insert before an emergency occurs! EpiPen® and EpiPen® Jr Auto-Injectors are disposable, prefilled automatic injection devices for use during allergic emergencies. They contain a single dose of epinephrine which you inject into your outer thigh. EpiPen® and EpiPen® Jr Auto-Injectors contain **no latex**. EpiPen® and EpiPen® Jr Auto-Injectors are intended for people who have been prescribed this medication by their physician.

It's important that you have this emergency medicine with you at all times. If you need additional units to keep at work, school, etc., please talk to your doctor. This information does not take the place of talking with your doctor about your medical condition or your treatment.

What is the most important information I should know about the EpiPen®/EpiPen® Jr Auto-Injector?

When you have an allergic reaction (anaphylaxis) use the EpiPen® or EpiPen® Jr Auto-Injector right away and immediately go to your doctor or emergency room for more medical treatment.

• It is important not to be afraid to use the EpiPen® or EpiPen® Jr Auto-Injector for emergency treatment of allergic reactions (anaphylaxis). For most people, injection of the EpiPen® or EpiPen® Jr Auto-Injector in the thigh does not hurt, and use of the EpiPen® or EpiPen® Jr Auto-Injector early at the start of such allergic reactions is important to help prevent the allergic reaction from becoming worse.

• Inject the EpiPen® or EpiPen® Jr Auto-Injector into the middle of the outer side of the thigh (upper leg).

• Epinephrine, the active ingredient in EpiPen® and EpiPen® Jr Auto-Injectors, is used to treat life-threatening allergic reactions (anaphylaxis). You should use this medication only if your doctor has prescribed it for allergic emergencies. Such emergencies may occur from insect stings or bites, foods, drugs, latex, other allergens, exercise induced anaphylaxis, or unknown causes.

• Make sure to tell your doctor about all your medical conditions and allergies.

• Always get medical treatment immediately after using the EpiPen® or EpiPen® Jr Auto-Injector.

Since you cannot predict when a life-threatening allergic reaction may occur, carry the EpiPen® or EpiPen® Jr Auto-Injector with you at all times.

What is the EpiPen®/EpiPen® Jr Auto-Injector?

The EpiPen® and EpiPen® Jr Auto-Injectors are products used for the emergency injection of epinephrine. Epinephrine is a medicine used for life-threatening allergic reactions such as severe swelling, breathing problems, or loss of blood pressure. Allergic reactions can be caused by stinging and biting insects, allergy injections, foods, medicines, exercise, or unknown causes.

Life-threatening allergic reactions may show up as closing of your breathing airways, wheezing, sneezing, hoarseness, hives, itching, swelling, skin redness, fast heartbeat, weak pulse, feeling very anxious, confusion, stomach pain, losing control of urine or bowel movements (incontinence), faintness, or “passing out” (unconsciousness).

The EpiPen® Auto-Injector (0.3 mg) is generally intended for patients who weigh 66 pounds or more (30 kilograms or more).

The EpiPen® Jr Auto-Injector (0.15 mg) is generally intended for patients who weigh approximately 33 to 66 pounds (15 to 30 kilograms).

Who should not use the EpiPen®/EpiPen® Jr Auto-Injector?

There are no absolute contraindications to the use of the EpiPen® or EpiPen® Jr Auto-Injector in a life-threatening allergic reaction. People with certain medical conditions have a higher chance of having serious side effects from the EpiPen® or EpiPen® Jr Auto-Injector.

Tell your doctor and pharmacist about all your medical conditions, but especially if you:

- Have heart disease or high blood pressure
- Have diabetes
- Have thyroid conditions
- Are pregnant

Tell your doctor about all the medicines you take, including prescription and non-prescription medicines, vitamins and herbal supplements. Inform your doctor of all known allergies. Some medicines may cause serious side effects if taken while you use the EpiPen® or EpiPen® Jr Auto-Injector. Some medicines may affect how the EpiPen® or EpiPen® Jr Auto-Injector works.

The EpiPen® or EpiPen® Jr Auto-Injector may affect how your other medicines work.

What should I avoid while using the EpiPen®/EpiPen® Jr Auto-Injector?

- **NEVER PUT THUMB, FINGERS OR HAND OVER ORANGE TIP. NEVER PRESS OR PUSH ORANGE TIP WITH THUMB, FINGERS OR HAND.** The needle comes out of orange tip. Accidental injection into finger, hands or feet may result in loss of blood flow to these areas. If this happens, go immediately to the nearest emergency room.
- Do not inject the EpiPen® or EpiPen® Jr Auto-Injector into the buttock or any other part of the body, other than the middle of the outer side of your thigh (upper leg).
- Do not inject EpiPen® or EpiPen® Jr Auto-Injector into a vein.
- Do not drop carrier tube or auto-injector. If carrier tube or auto-injector is dropped, inspect for damage and leakage. Discard auto-injector and carrier tube, and replace if damage or leakage is noticed or suspected.

What are the possible side effects of the EpiPen®/EpiPen® Jr Auto-Injector?

Too much epinephrine can cause dangerous high blood pressure or stroke.

If you take certain medicines, you may develop serious life-threatening side effects from the epinephrine in the EpiPen®/EpiPen® Jr Auto-Injector. Be sure to tell your doctor all the medicines you take, especially medicines for asthma.

Patients with certain medical conditions, or who take certain medicines, may get more side effects from the EpiPen® or EpiPen® Jr Auto-Injector or the side effects may last longer. This includes patients who take certain types of medicines for asthma, allergies, depression, low thyroid, high blood pressure, and heart disease. Patients with heart disease may feel chest pain (angina).

The EpiPen® or EpiPen® Jr Auto-Injector (epinephrine) can cause the following reactions. Some reactions can be serious. They usually go away with rest. **Please notify your doctor if you experience any of these.**

Common side effects of the EpiPen® or EpiPen® Jr Auto-Injector include:

- Faster, irregular (wrong) or “pounding” heartbeat
- Sweating
- Nausea and vomiting
- Breathing problems
- Paleness
- Dizziness
- Weakness or shakiness
- Headache
- Feeling of over excitement, nervousness or anxiety

These are not all the possible side effects of the EpiPen® or EpiPen® Jr Auto-Injector.

For more information, ask your doctor or pharmacist. Call your doctor for medical advice about side effects. You may report side effects to FDA at 1-800-FDA-1088.

How should I store the EpiPen®/EpiPen® Jr Auto-Injector?

- Keep the EpiPen® or EpiPen® Jr Auto-Injector nearby and ready for use at all times.
- Store at 25°C (77°F); excursions permitted to 15°C-30°C (59°F-86°F) (See USP Controlled Room Temperature). Contains no latex. Protect from light.
- **Do NOT** store in refrigerator.
- **Do NOT** expose to extreme cold or heat. For example, do **NOT** store in your vehicle's glove box.
- Examine contents in clear window of auto-injector periodically. If the solution is discolored or contains solid particles (precipitate), replace the unit. Solution should be clear.
- **Always keep your EpiPen® or EpiPen® Jr Auto-Injector in the carrier tube with the blue safety release on until you need to use it.**


Your auto-injector has an expiration date.

- Example: “ DEC 11 ” = December 31, 2011
- Replace it before the expiration date.

SEE OTHER SIDE FOR "DIRECTIONS FOR USE"

DIRECTIONS FOR USE

- **REMOVE AUTO-INJECTOR FROM CARRIER TUBE BEFORE USE.**
- **NEVER PUT THUMB, FINGERS OR HAND OVER ORANGE TIP.**
- **NEVER PRESS OR PUSH ORANGE TIP WITH THUMB, FINGERS OR HAND.**
- **THE NEEDLE COMES OUT OF ORANGE TIP.**
- **DO NOT REMOVE BLUE SAFETY RELEASE UNTIL READY TO USE.**
- **DO NOT USE IF SOLUTION IS DISCOLORED.**
- **DO NOT PLACE PATIENT INSERT OR ANY OTHER FOREIGN OBJECTS IN CARRIER WITH AUTO-INJECTOR, AS THIS MAY PREVENT YOU FROM REMOVING THE AUTO-INJECTOR FOR USE.**

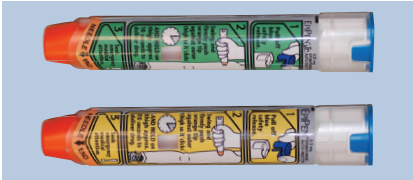

TO REMOVE AUTO-INJECTOR FROM THE CARRIER TUBE:



1. Flip open the yellow cap of the EpiPen® or the green cap of the EpiPen® Jr Auto-Injector carrier tube.



2. Remove the EpiPen® or EpiPen® Jr Auto-Injector by tipping and sliding it out of the carrier tube.

TO USE AUTO-INJECTOR:



1. Grasp unit with the orange tip pointing downward.



2. Form fist around the unit (orange tip down).
3. With your other hand, pull off the blue safety release.

4. Hold orange tip near outer thigh.

DO NOT INJECT INTO BUTTOCK.



(Auto-injector is designed to work through clothing.)



5. Swing and **firmly push** against outer thigh until it clicks so that unit is perpendicular (at 90° angle) to the thigh.
6. Hold **firmly against thigh** for approximately 10 seconds to deliver drug. (The injection is now complete. The window on auto-injector will be obscured.)
7. Remove unit from thigh (the orange needle cover will extend to cover needle) and massage injection area for 10 seconds.
8. Call 911 and seek immediate medical attention.
9. Take the used auto-injector with you to the hospital emergency room.

Note: Most of the liquid (about 85%) stays in the auto-injector and cannot be reused. However, you have received the correct dose of the medication if the orange needle tip is extended and the window is obscured. Trainer label has blue background color. Blue background labeled trainer contains no needle and no drug.

! WARNING !

- **NEVER** put thumb, fingers or hand over orange tip. **NEVER** press or push orange tip with thumb, fingers or hand. The needle comes out of orange tip. Accidental injection into hands or feet may result in loss of blood flow to these areas. If this happens, go immediately to the nearest emergency room.
- The EpiPen® and EpiPen® Jr Auto-Injector should be injected only into the outer thigh (see "Directions for Use"). **DO NOT INJECT INTO BUTTOCK.**
- Do **NOT** remove blue safety release until ready to use.

To dispose of expired units

- Expired auto-injectors must be disposed of properly.
- To dispose of an expired auto-injector and carrier tube, take them to your doctor's office or to a hospital for proper disposal.
- Used auto-injector with extended needle cover will not fit in carrier tube.

IMMEDIATELY AFTER USE

- **Go immediately to the nearest hospital emergency room or call 911.**

You may need further medical attention. Take your used auto-injector with you.

- Tell the doctor that you have received an injection of epinephrine in your thigh.
- Give your used EpiPen® or EpiPen® Jr Auto-Injector to the doctor for inspection and proper disposal.

Do not attempt to take the auto-injector apart.

Manufactured for: Dey Pharma, L.P., Napa, CA 94558 USA
by Meridian Medical Technologies™, Inc., Columbia,
MD 21046 USA, a subsidiary of King Pharmaceuticals®, Inc.

EpiPen® and EpiPen® Jr are registered trademarks of Mylan, Inc.
licensed exclusively to its wholly-owned affiliate, Dey Pharma,
L.P. of Napa, California, USA
©2011 by Meridian Medical Technologies™, Inc.
06/2011 03-855-02 0001697

EpiPen®
(Epinephrine) Auto-Injector 0.3 mg

EpiPen® Jr
(Epinephrine) Auto-Injector 0.15 mg

MyEpiPen.com

Register your EpiPen®/EpiPen® Jr at
MyEpiPen.com and find out more about:

- **EpiPen® REFILL REMINDER PROGRAM**
It is important to keep your auto-injector up-to-date. Sign up now for this FREE program.
- Register up to 6 EpiPen®/EpiPen® Jr Auto-Injectors and receive automatic REFILL REMINDER ALERTS.
- Sign up to receive periodic information related to allergies and allergens.
- Instructional Video

For more information about EpiPen®/
EpiPen® Jr and proper use of the product,
visit us at **EpiPen.com**.